## NPHL Reportable Disease Quick Shipping Guide - Culture Dependent Methods

## **Immediate Notification:**

Douglas Co. (402) 444-7214 (after hrs (402) 444-7000)

Lancaster Co. (402) 441-8053 (after hrs (402) 441-8000)

All Other Counties (402) 471-2937 (goes to Poison Center after hours)

Nebraska Public Health Laboratory (402) 559-9444 for possible select agents and specimen collection/shipping questions

	Call				
	Immediately?	What to send?	How to ship?	What to order on NPHL form?	What to order in NUlirt?
Carbapenem-Resistant Acinetobacter baumanii	-	Original culture plate	Category B	Antibiotic Resistance to Confirm	CARBAR
Bacillus anthracis*	YES	Original culture plate	Category A	BTID - Report on STATPack	BTID
Bordetella pertussis		Charcoal Agar-isolate	Category B	Rpt Confirmed/Bank only	BPERT or BPD
Brucella abortus, B. melitensis, and B. suis*	YES	Original culture plate	Category A	BTID - Report on STATPack	BTID
Burkholderia (Pseudomonas) mallei and B.pseudomallei *	YES	Original culture plate	Category A	BTID - Report on STATPack	BTID
Carbapenem-Resistant Enterobacterailes (suspected or confirmed)		Isolate	Category B	Antibiotic Resistance to Confirm	CARBAR
E. coli O157-H7 and other Shigatoxin-postive E. coli			Category A	Rpt Confirmed/Bank only	NPHLBK
Shigatoxin positive stools (E. coli) from GN broth		Original Stool in Maintenance Media (do not send GN)	Category B	Stool positive by EIA	HECCU
Francisella tularensis*	YES	Original culture plate	Category A	BTID - Report on STATPack	BTID
Haemophilus influenzae (sterile site only)	YES	Isolate	Category B	Serogroup or Serotype isolate	NPHLBK
Listeria monocytogenes	YES	Isolate	Category B	Rpt Confirmed/Bank only	NPHLBK
Neisseria meningitidis (sterile site only)	YES	Isolate	Category B	Serogroup or Serotype isolate	BNK
Mycobacterium spp, invasive infection (including probable M. tuberculosis complex).		LJ Slant	Category A	Mycobacteriology/TB	AFBCU, AFBIS, TBAPR, TBGEN, AFBIS
Mycobacterium spp, invasive infection (including atypical Mycobacterium spp or slow growing MAI group).		LJ Slant	Category B	Mycobacteriology/TB	AFBCU, AFBIS, TBAPR, TBGEN, AFBIS
Salmonella serogroups, including serotype Typhi; Send isolates requiring serogrouping to your reference lab.	Yes Salmonella spp. serotype Typhi ONLY	Isolate (serotype Typhi only)	Category B	Rpt Confirmed/Bank only	NPHLBK
Shigella spp*** (Send isolates requiring serogrouping to your reference lab. Forward all species except S.sonnei to NPHL. Notify NPHL if speciated as S. dysenteriae for special Category A shipping requirements)		Isolate	Category B	Rpt Confirmed/Bank only	NPHLBK
Staphylococcus aureus, vancomycin-intermediate/resistant (MIC≥4mg/mL)		Isolate	Category B	Antibiotic Resistance to Confirm	BNK or ORGCU
Streptococcus pneumoniae (sterile sites only)		Isolate	Category B	Rpt Confirmed/Bank only	NPHLBK
Vibrio cholera	YES		Category B	Rpt Confirmed/Bank only	NPHLBK
Vibrio spp., not V. cholera		Isolate	Category B	Rpt Confirmed/Bank only	NPHLBK
Yersinia pestis*	YES		Category A	BTID - Report on STATPack	BTID
Yoraini-2-9992099t Y. pestis		Isolate	Category B	Rpt Confirmed/Bank only	NPHLBK

See Footnotes on back							
* Designated as potential select agents by CDC							
CRF/ CRPA/ CRAB Submission requirements for NPHI							

- Submit a fresh subculture (<18 hours) to avoid delay in testing
- Submit isolates of Enterobacterales and/ or Pseudomonas aeruginosa and/or Acinetobacter baumanii that are non-susceptible (intermediate or resistant) to carbapenems as mentioned below:
  - ⊙ Enterobacterales: Ertapenem MIC ≥ 1 μg/ml or meropenem MIC ≥ 2 μg/ml or imipenem MIC ≥ 2 μg/ml or non-susceptible by disc diffusion method (See rare exceptions below)
  - Pseudomonas aeruginosa: Meropenem or imipenem MIC ≥ 4 µg/ml, or non-susceptible by disc diffusion method and non-susceptible to both Cefepime and Ceftazidime MIC ≥ 16 µg/ml
  - Acinetobacter baumanii: Doripenem ≥ 4 µg/ml or Imipenem ≥ 4 µg/ml or Meropenem ≥ 4 µg/ml or non-susceptible by disc diffusion method
  - Submit all isolates of in-house or reference laboratory confirmed carbapenemase-producing Enterobacterales (CPE) or Pseudomonas aeruginosa (CP-PA) or Acinetobacter baumanii (CRAB)
- Exceptions:
  - DO NOT submit the following isolates
    - Proteus species, Providencia species, and Morganella morganii non-susceptible only to imipenem but susceptible to meropenem and ertapenem
    - Pseudomonas aeruginosa isolates that are mucoid or from cystic fibrosis patients
- \*\*\* Candida auris may be misidentified using some phenotypic methods. Misidentification can lead to inappropriate patient treatment and delay appropriate infection control precautions. If these Candida spp are identified by the following systems, send isolates to NPHL to confirm. https://www.cdc.gov/fungal/candida-auris/pdf/Testing-algorithm\_by-Method 508.pdf
  - Vitek 2 YST Candida haemulonii or Candida duobushaemulonii
  - API 20C Candida sake, Rhodotorula glutinis (characteristic red color not present)
  - API ID 32C Candida intermedia, Candida sake, Saccharomyces kluyveri
  - BC Phoenix yeast identification system Candida haemulonii, Candida catenulata
  - MicroScan Candida famata, Candida quilliermondii, Candida lusitaniae, Candida parapsilosis
  - RapID Yeast Plus Candida parapsilosis

\*\*Susceptibilities on Campylobacter and Shigella spp are recommended in-house or by commercial reference laboratory. Send isolates to NPHL only when specially requested. Immediate Notification for automated ELR Labs - Required to call by telephone to a live public health surveillance official within 24 hours of detection Laboratories must submit the isolate and/or specimen within 7 days to the Nebraska Public Health Lab Category B - NPHL UN3373 Shipping Box Category A - NPHL UN2814 Shipping Box, including Shipper's Declaration. Shipper required to be certified every 2 years

NOTE: Future updates will be posted on http://nphl.org . Periodically reprint and post on bench for most accurate information.